Kraftwerk[®]

DE

EN

FR

ES

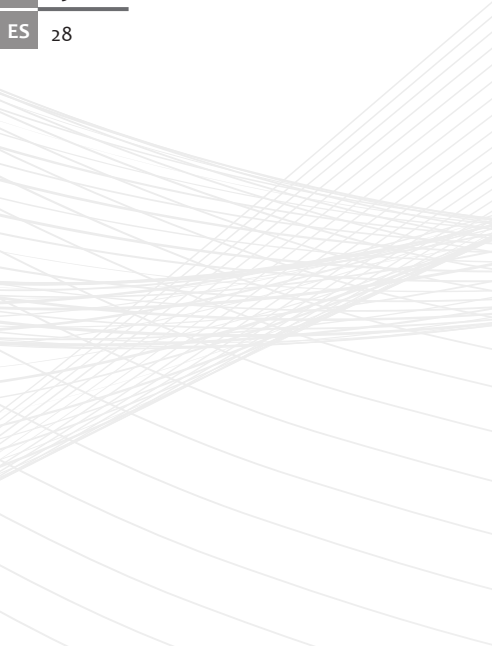
Kraftwerk[®]

Massage Pistole

powered by axion

DE	1
EN	10
FR	19
ES	28

Kontraindikationen

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<https://www.ear-system.de/ear-verzeichnis/sammel-und-ruecknahmestellen.jsf>

Contraindications

Please consult a physician before use if you are in any of the following situations.

1. If you are pregnant or have recently given birth.
2. If you have heart disease or have a pacemaker implanted.
3. If you have fractures, osteoporosis or an abnormal spine or joints.
4. If you suffer from a malignant tumor and are being treated with chemotherapy or radiation therapy.
5. Please do not use if you suffer from hemophilia.
6. Avoid massaging areas with local skin infections, deep bruising or varicose veins.
7. If you have suffered bruises or feel painful and uncomfortable, do not use the device.
8. If you suffer from or are prone to thrombosis, you should always consult your physician before using the device. You should also do so if you suffer from any medical condition not listed above.

Safety information

1. Children should not use the device.
2. Do not use the device on damp, wet or unclean skin.
3. Do not use it on the head area, on body parts with less muscles, directly on your spine or on your bones. Please massage with the device only body parts with soft tissue and without feeling pain or discomfort.

4. Please do not insert any objects into the ventilation openings of the device.
5. Do not immerse the device in water. Also, do not allow water to enter the ventilation openings.
6. Please do not use this product while using spraying products.
7. Do not use this product in humid places such as bathrooms.
8. Do not place the device near a fire source.
9. Do not intentionally drop the device.
10. Do not open or disassemble the device.
11. Do not allow your fingers, hair or other parts to get near the opening for the massage head connection.

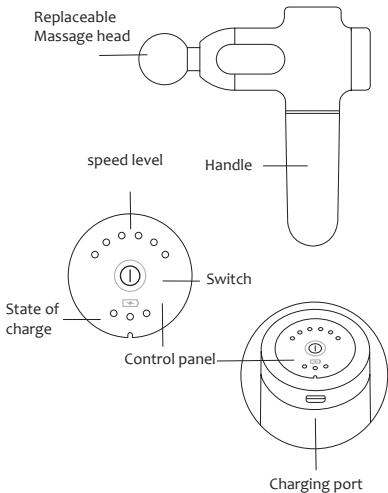
Warnings

1. Please unplug the charger after charging.
2. Please use the massage head that gives you the best massage effect.
3. Please do not use it for more than 15 minutes. If you want to continue to use it, please let it rest for 30 minutes, then you can use it again.
4. Please use a charger with 5V $\overline{\text{---}}$ 2A or more to charge it.
5. Please do not leave the device charging unattended.
Unplug the device when you leave the house.

Product description

EN


Application of the exchangeable heads

EN

It is recommended to use the device on your clothes. Gently press and move the device for about 60 seconds per body part.

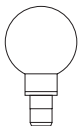Round massage head


The soft and least invasive head shape is suitable for general use on all parts of the body, making it ideal for beginners to learn the effects of the massage gun. The head is mainly used for large muscle groups such as buttocks, hamstrings and thighs, as it has a larger surface area. In the slower speed settings, it is ideal for massaging the neck.


Bullet massage head

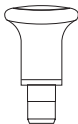This head is ideal for trigger point massage. It concentrates the impulses on a small area and is therefore ideal for a massage with depth effect or for the application on hands and feet.

Flat massage head

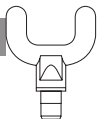The flat head has a firm, smooth, flat surface. It is very suitable for acting on larger muscle groups, such as thighs, buttocks, calves and pectorals. This is also an ideal multi-purpose attachment for relaxing and shaping muscle groups.



Fork massage head

This head is often used in the area of the Achilles tendon. You can place the Achilles tendon between the fork. The attachment does not exert pressure on the tendon itself, but treats the surrounding tissue.


Use/change of the head

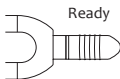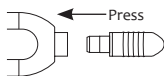Please turn off the massage gun and slowly insert the massage head into the groove of the muzzle. When they are joined, the insertion of the head is completed.

Operating instructions

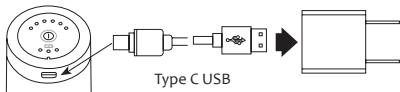Notes on the charging process

The power supply unit is not included in the scope of delivery. Please use your own power adapter. Note: Use e.g. a cell phone charger with 5V $\overline{=}$ 2A or more.

1. Please fully charge the massage gun for about four hours before first use.
 2. The charging indicator flashes regularly. It indicates that the massage gun is being charged.
 3. When all charging indicators light up, the massage gun is fully charged.
 4. If the charging indicator on the far left flashes, the massage gun will soon be empty. Please charge it in time.
- The massage gun can be charged at any time and at any charge level.
 - The massage gun is not ready for operation during charging. It must be disconnected from the mains before use.
 - Please fully charge the battery every three months if the device is not used for a long time.

Usage notes

Turn on / off:

Press the button for 2 seconds to switch the device on or off.

Speed setting:

1. By default, the massage gun runs at the first speed level when switched on (the first level indicator lights up). Press briefly to set the speed. There are six speed levels in total. When the sixth speed level is reached, press briefly to return to the slowest speed level.
2. In each speed level, you can long press the button to turn off the massage gun.
3. After 15 minutes of continuous operation, the intelligent automatic shut-off will kick in. If you want to use the massage gun again, you have to switch it on again.

Cleaning and maintenance

Please remember to switch off the massage gun before cleaning.

1. Please clean it with a slightly damp towel and then dry it with a soft cloth.
2. Please do not use detergents, cleaning brushes, gasoline, kerosene, glass cleaner or thinner to clean.
3. Please keep it in the original package or put it in a safe, dry, cool and ventilated place after wiping off dust and dirt.
4. The massage gun is designed and manufactured in such a way that it is not necessary to maintain or lubricate it regularly.

Troubleshooting

Problem	Cause/ solution
Device does not work	Please check if the device is charged.
Device stops during the massage	<ol style="list-style-type: none">1. The battery is not charged.2. The switch-off key was pressed by mistake.3. The device has a 15-minute automatic switch-off. The device therefore switches off automatically after 15 minutes.
Quiet noise is audible	Please take into account that the mechanics of the device makes quiet noises. It is not a malfunction.

Technical data

Product name: KRAFTWERK Massage-Gun
(Article-No. 30500)
Rated voltage: DC 5V Rated power: 25W

EN

Part list

1x KRAFTWERK Massage-Gun
4x Massage heads
1x Charging cable (Typ C USB)
1x Manual

Disposal

Follow local regulations when disposing of materials. Dispose of the device in accordance with the Waste Electrical and Electronic Equipment (WEEE) Regulation 2002/96/EC. If you have any questions, please contact the local authority responsible for disposal

EAR Foundation

As a manufacturer in the sense of the ElektroG we are registered at the responsible foundation Elektro-Altgeräte Register (Benno-Straß-Str. 1, 90763 Fürth).

Electrical and electronic equipment

EN

Information for private households pursuant to Section 18 (2) of the Electrical and Electronic Equipment Act (ElektroG)

The Electrical and Electronic Equipment Act (ElektroG) contains a large number of requirements for the handling of electrical and electronic equipment. The most important ones are summarized here.

1. Separate collection of waste equipment

Electrical and electronic equipment that has become waste is referred to as old equipment. Owners of old devices have to collect them separately from unsorted municipal waste. Old devices do not belong in particular into the domestic refuse, but into special collecting and return systems.

2. Possibilities for the return of old appliances

Owners of old appliances from private households can hand them in at the collection points of the public waste management authorities or at the take-back points set up by manufacturers or distributors within the meaning of the ElektroG. An online directory of collection and return points can be found here:
<https://www.ear-system.de/ear-verzeichnis/sammel-und-ruecknahmestellen.jsf>

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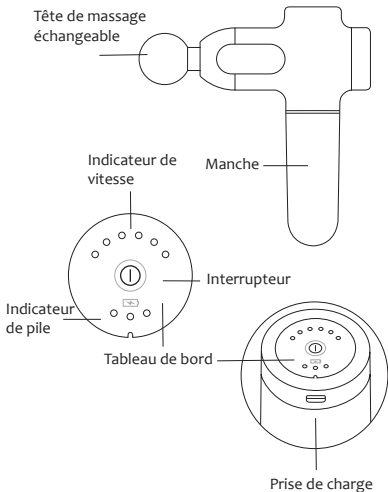
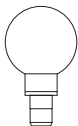
It is recommended to use the device on your clothes.
Gently press and move the device for about 60 seconds
per body part.

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axion GmbH